



The strength of a dermal template.

The simplicity of collagen.

## 1 Prepare



Cleanse the wound bed.

## 2 Apply




Trim dressing to fit the wound.

If needed, multiple sheets can be used to cover the surface of the wound.

Place the dry dressing in the wound and hydrate with exudate or saline solution until moistened.

## 3 Protect

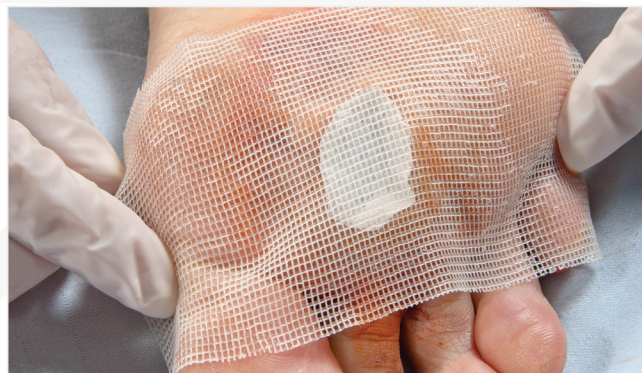


The Endoform dermal template may need protection from adhering to the secondary dressing.

Cover the Endoform dermal template with a non-adherent dressing.

Example: Restore Contact Layer FLEX dressing.

## 4 Cover

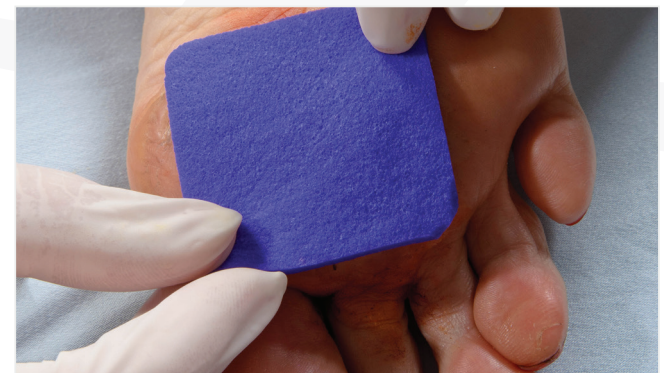

Select appropriate secondary dressing based on type of wound and amount of exudate.

If little to no exudate, apply a hydrogel to maintain appropriate moisture balance.

Example: Hydrofera Blue dressing (shown) or Restore border foam dressing with Triact technology.

## 5 Secure



If needed, secure cover dressing with rolled gauze.

The Endoform dermal template may be used in conjunction with compression therapy and negative pressure wound therapy.

## 6 Reapply



Reapply the Endoform dermal template every 5-7 days or as needed.

It is not necessary to remove any residual Endoform dermal template during dressing changes or reapplication of product.



Manufactured for:  
**Hollister Wound Care**  
2000 Hollister Drive  
Libertyville, IL 60048 USA  
1.888.740.8999

[www.hollisterwoundcare.com](http://www.hollisterwoundcare.com)

Distributed in Canada by:  
**Hollister Limited**  
95 Mary Street  
Aurora, Ontario L4G 1G3  
1.800.263.7400

**R** Caution: Federal (USA) law restricts this device for sale by or on the order of a physician or licensed healthcare professional. Refer to Instructions for Use for contraindications, warnings, precautions, and possible complications.

For product questions, sampling needs, or detailed clinical questions concerning our products in the US call **1.888.740.8999**. In Canada call **1.800.263.7400**.

HCPCS are for reference only and subject to change  
Hollister and logo is a trademark of Hollister Incorporated.  
Endoform is a trademark of Mesynthes Limited.  
©2013 Hollister Incorporated.  
922350-914